
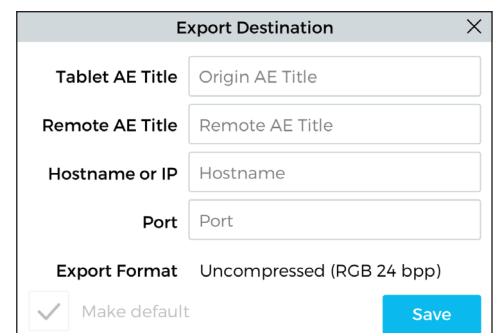
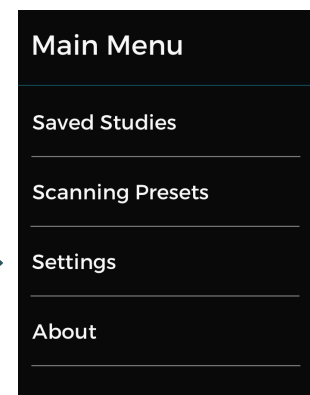
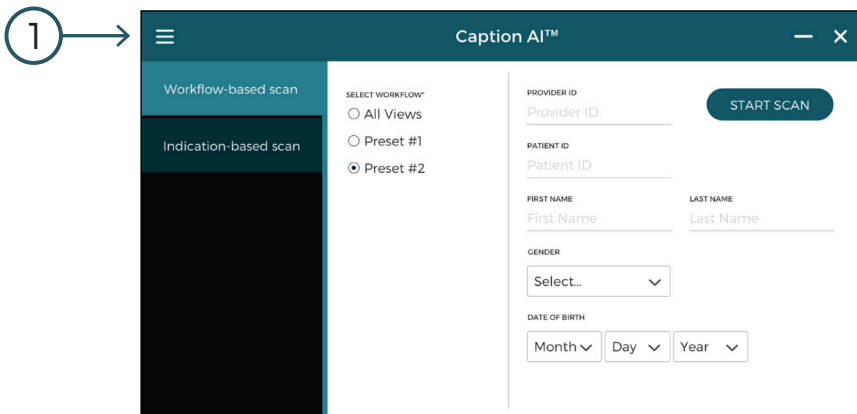


## Caption AI

# Configuring Caption AI to send studies to PACS

Thank you for choosing Caption Health. This guide will help you configure Caption AI software settings for exporting studies to PACS.

1. Tap  to open the **Main Menu**
2. Select **Settings**
3. Under **Export**, tap the **“Add”** button
4. Enter destination information and select “Save”



# Caption Health

**Note:** To set up the MAC address, first type “CMD” in the Terason search field. Then, in the terminal, type **ipconfig/all** to locate the Terason MAC address information.

To identify the MAC address in the terminal (for a **wired** or **wireless** adapter), locate the following fields in the output:

```
C:\Users\terason>ipconfig/all

Windows IP Configuration

Host Name . . . . . : TERASON-3-DVT-16
Primary Dns Suffix . . . . . :
Node Type . . . . . : Hybrid
IP Routing Enabled. . . . . : No
WINS Proxy Enabled. . . . . : No
DNS Suffix Search List. . . . . : hsd1.ca.comcast.net

Ethernet adapter Ethernet:

Connection-specific DNS Suffix . : hsd1.ca.comcast.net
Description . . . . . : Intel(R) T211 Gigabit Ethernet Controller
Physical Address. . . . . : 00-18-7D-BE-A3-02
DHCP Enabled. . . . . : Yes
Autoconfiguration Enabled . . . . : Yes
IPv6 Address. . . . . : 2601:644:103:1f60::7b74(Preferred)
Lease Obtained. . . . . : Tuesday, April 14, 2020 8:42:41 AM
Lease Expires . . . . . : Monday, April 20, 2020 8:59:09 AM
IPv6 Address. . . . . : 2601:644:103:1f60:812f:38c5:2c20:5f4a(Preferred)
IPv6 Address. . . . . : 2601:644:103:1f64:812f:38c5:2c20:5f4a(Preferred)
Temporary IPv6 Address. . . . . : 2601:644:103:1f60:14be:782b:d96d:af7e(Preferred)
Temporary IPv6 Address. . . . . : 2601:644:103:1f64:14be:782b:d96d:af7e(Preferred)
Link-local IPv6 Address . . . . . : fe80::812f:38c5:2c20:5f4a%18(Preferred)
IPv4 Address. . . . . : 10.0.0.175(Preferred)
Subnet Mask . . . . . : 255.255.255.0
Lease Obtained. . . . . : Tuesday, April 14, 2020 8:42:39 AM
Lease Expires . . . . . : Tuesday, April 21, 2020 8:42:40 AM
Default Gateway . . . . . : fe80::fe91:14ff:fecf:44c8%18

Wireless LAN adapter Wi-Fi:

Connection-specific DNS Suffix . : hsd1.ca.comcast.net
Description . . . . . : Realtek 802.11a/b/g/n Wireless LAN Adapter
Physical Address. . . . . : 80-C5-F2-C1-63-37
DHCP Enabled. . . . . : Yes
Autoconfiguration Enabled . . . . : Yes
IPv6 Address. . . . . : 2601:644:103:1f60::6469(Preferred)
Lease Obtained. . . . . : Tuesday, April 14, 2020 9:11:27 AM
Lease Expires . . . . . : Tuesday, April 21, 2020 9:11:28 AM
IPv6 Address. . . . . : 2601:644:103:1f60:4163:706b:9fbc:58ad(Preferred)
Temporary IPv6 Address. . . . . : 2601:644:103:1f60:9c2d:952e:01ba:a456(Preferred)
Link-local IPv6 Address . . . . . : fe80::4163:706b:9fbc:58ad%11(Preferred)
IPv4 Address. . . . . : 10.0.0.127(Preferred)
Subnet Mask . . . . . : 255.255.255.0
Lease Obtained. . . . . : Tuesday, April 14, 2020 9:11:25 AM
Lease Expires . . . . . : Tuesday, April 21, 2020 9:11:25 AM
Default Gateway . . . . . : fe80::fe91:14ff:fecf:44c8%16
10.0.0.1
DHCP Server . . . . . : 10.0.0.1
DHCPv6 IAID . . . . . : 310429170
DHCPv6 Client DUID. . . . . : 00-01-00-01-24-D3-7E-70-00-18-7D-BE-A3-02
DNS Servers . . . . . : 2001:558:feed::1
75.75.75.75
75.75.76.76
```

Questions? We're here to help.

## URGENT

Call (855) 506-4357 8am - 8pm ET (US)

## NON-URGENT

Text (855) 506-4357 Preferred - 24/7 support

Email [support@captionhealth.com](mailto:support@captionhealth.com)