

Caption Health

Congratulations on upgrading Caption AI! The Caption Health team is constantly working to build out features, refine algorithms, and make Caption AI easier to use.

Welcome to **Caption AI** Version 0.10.0

Here's **what's new** in this version:

> **Custom Protocols & Dynamic Scanning**

- Users now have the flexibility to create and follow a customizable workflow-based protocol or indication-based protocol
- Caption AI gives users the ability to skip to different views in the protocol when scanning
- Users can also play recorded clips during the acquisition flow
- Users can now scan with the general imaging convention

> **Prescriptive Guidance**

- Improvements to existing algorithms mean that Caption Guidance is more responsive than ever
- Users get more real-time, responsive feedback with 90+ types of guidance (compared to 50 in the previous version)
- New, intuitive icons make it easier for users to understand what kinds of probe movements to make
- Even when structures are not visualized, users receive guidance on probe movements to try

> **Quality Meter**

- Quality assessment is now more closely aligned with an expert's assessment of image quality

> **Save Best Clip and Auto-Capture**

- A new metric shows users how close they have gotten to the Auto-Capture threshold so that they can make an informed decision on when to use the Save Best Clip option

Caption Health

- The Save Best Clip option now comes up sooner so users have the option to use it when they need it
- Users also have the flexibility to turn off Auto-Capture

> Starting Visual Guidance

- Updated probe positioning visuals for certain views (i.e., apical views) now show patient positioning
- Additional written guidance for probe positioning, updated language for patient positioning, and updated default depth levels make it easier to start scanning in the right place

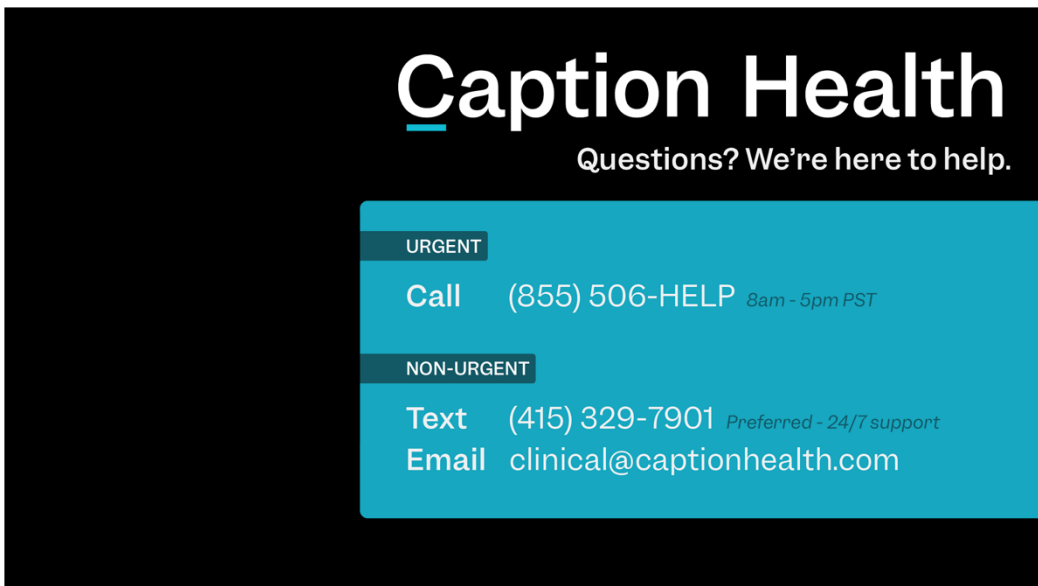
> Support & Security

- UDI numbers are now produced and display on the About page
- If an incompatible probe is plugged, users will receive an error

Other Tips:

- Reference the Quick Start Guide and other documents provided
- While a study is exporting, avoid pressing any buttons on the screen until the export is complete

Still experiencing issues?

A graphic with a black background and a teal contact box. The text 'Caption Health' is in white with a blue underline under the 'C'. Below it, 'Questions? We're here to help.' is in white. The teal box contains contact information for urgent and non-urgent issues.

Caption Health
Questions? We're here to help.

URGENT
Call (855) 506-HELP 8am - 5pm PST

NON-URGENT
Text (415) 329-7901 Preferred - 24/7 support
Email clinical@captionhealth.com